

AAJT-S

ABDOMINAL AORTIC AND JUNCTIONAL TOURNIQUET STABILIZED



INDICATION FOR USE

To control bleeding in the pelvis, inguinal area, axilla, and for pelvic fracture stabilization

RELATIVE CONTRAINDICATIONS FOR USE IN ABDOMINAL PLACEMENT

Known Abdominal Aortic Aneurysm
Pregnancy

NO CONTRAINDICATIONS FOR INGUINAL OR AXILLARY PLACEMENT

WARNINGS

- If the product is damaged, do not use
- Removal may lead to resumed bleeding

RECOMMENDED TIME OF APPLICATION

4 hours - Junctional placement
2 hours - Abdominal placement



Scan the QR code to watch
a how-to-use video.





AXILLA



GROIN | INGUINAL



PELVIC BLEEDING

STEP ONE

Secure the device around the patient and connect the ladder strap until **RED** meets **RED**.

**STEP TWO**

Position the bladder over the target site.

**STEP THREE**


Tighten belt - REMOVE ALL SLACK.

**STEP FOUR**

Use the ratcheting buckle to complete tightening. (3-5 clicks)

**STEP FIVE**

Inflate bladder with hand bulb until **GREEN** indicator shows on pressure valve.

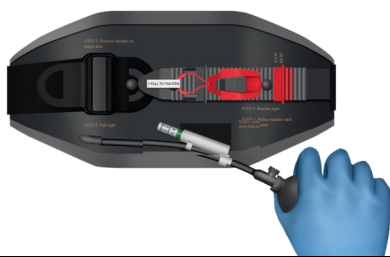**JUNCTIONAL PLACEMENT**

(AXILLA | GROIN | INGUINAL)

UP TO 4 HOURS

ABDOMINAL PLACEMENT

PELVIC BLEEDING | BILATERAL GROIN

UP TO 2 HOURS



PELVIC STABILIZATION FOUR STEPS

PELVIC STABILIZATION

STEP ONE

Secure the device around the patient's greater trochanters and connect the ladder strap until **RED** meets **RED**.



STEP TWO

Position the bladder along the midline around the greater trochanters.



STEP THREE


Tighten belt - REMOVE ALL SLACK.



STEP FOUR

Use the ratcheting buckle to complete tightening. (3-5 clicks)

DO NOT INFLATE.

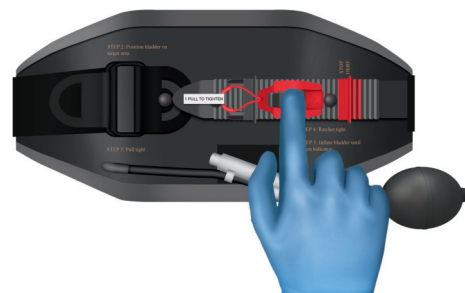

	Caution
	Consult instructions for use
	Single patient - multiple use
	Non-sterile
	Medical device
	Authorized Rep in the European Community
	Manufacturer
	CE Mark
	Federal law restricts this device to sale by or on the order of a physician



Compression Works Inc.
1634-A Montgomery Hwy #115
Birmingham, AL 35216 USA
www.compressionworks.com
Hemorrhage Stops Here™
NSN: 6515-01-6954457



Elara Pharmservices Europe Ltd
Office 107, Regus Block 1, Blanchardstown
Corporate Park, Dublin, Ireland, D15AKK1
AAJT-S IFU US 003 20251016