

AAJT-S APPLICATION & REMOVAL GUIDE

STEP-BY-STEP INSTRUCTIONS FOR SAFE AND EFFECTIVE USE
WWW.COMPRESSIONWORKS.COM



compression**works**


REMOVAL

1



DEFLATE THE
BLADDER SLOWLY.

2

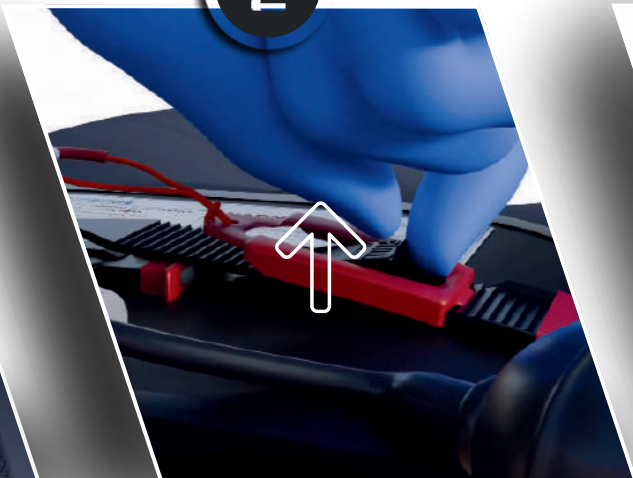

LIFT AND RELEASE
THE BLACK PULL TAB.

3



REMOVE THE
LADDER STRAP.

APPLICATION




1

CONNECT
RED TO RED.



2

POSITION
THE DEVICE

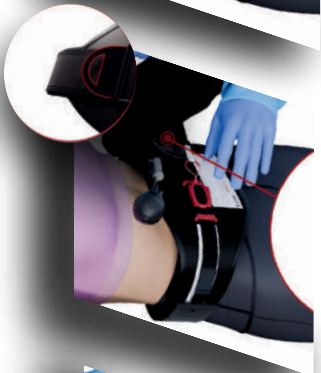

3

REMOVE SLACK
FROM BELT



4

RATCHET TO
TIGHTEN



5

INFLATE BLADDER
UNTIL **GREEN**

DO NOT CUT



PREMATURE REMOVAL
MAY RESULT IN
RESUMED BLEEDING



SCAN THE QR CODE
TO WATCH A
HOW-TO-USE VIDEO.